

**ZOLL R Series PLUS Competency Checklist #DAHS-NSCRSPLUS17**

Page 1 of 2

<b>Name:</b>	<b>PPS#:</b>
<b>Unit:</b>	<b>Title:</b>
PERFORMANCE CRITERIA - Unless otherwise specified all skills will be demonstrated in accordance with the appropriate UC Davis Health Policy and Procedure.	
<b>These skills will be considered complete when all below performance criteria are completed and checklist is scanned and emailed to <a href="mailto:hs-cppn@ucdavis.edu">hs-cppn@ucdavis.edu</a></b>	

1.	COMPLETED
Completed the assigned ZOLL R Series PLUS Defibrillator tutorials in UC Learning.	<input type="checkbox"/>
<b>TEST MODE</b>	
Successfully demonstrates 30 Joule defibrillator test.	<input type="checkbox"/>
Can check and change paper.	<input type="checkbox"/>
<b>ADVISORY MODE</b>	
Turn dial to ON.	<input type="checkbox"/>
Tells everyone to stop CPR and stand clear.	<input type="checkbox"/>
Follows voice prompts and delivers shock if recommended.	<input type="checkbox"/>
<b>MANUAL MODE</b>	
Can turn on device and convert from AED to manual mode.	<input type="checkbox"/>
<b>CPR FEEDBACK</b>	
Demonstrates steps to fill CPR Index™ – understands proper rate/depth.	<input type="checkbox"/>
Shows that if rate is too slow, metronome beeps and <u>Rate</u> prompt appears.	<input type="checkbox"/>
Speeds up to silence metronome and allow the <u>Rate</u> prompt to disappear.	<input type="checkbox"/>
Shows that if depth is too shallow, the <u>Depth</u> prompt appears on the screen.	<input type="checkbox"/>
Pushes hard to allow <u>Depth</u> prompt to disappear.	<input type="checkbox"/>
Demonstrates understanding of See-Thru CPR® filtered ECG.	<input type="checkbox"/>
<b>PADS</b>	
Connects OneStep™ pads to OneStep cable (or other pads/paddles if applicable).	<input type="checkbox"/>
Opens OneStep packaging correctly	<input type="checkbox"/>
Demonstrates proper pad placement for defibrillation, pacing, and cardioversion.	<input type="checkbox"/>
Identifies CPR Sensor and explains its purpose.	<input type="checkbox"/>

**ZOLL R Series PLUS Competency Checklist #DAHS-NSCRSPLUS17**  
Page 2 of 2

<b>Name:</b>	<b>PPS#:</b>
<b>Unit:</b>	<b>Title:</b>

PERFORMANCE CRITERIA - Unless otherwise specified all skills will be demonstrated in accordance with the appropriate UC Davis Health Policy and Procedure.  
**These skills will be considered complete when all below performance criteria are completed and checklist is scanned and emailed to [hs-cppn@ucdavis.edu](mailto:hs-cppn@ucdavis.edu)**

SUPERUSER/TRAINER	
Demonstrate how to use additional options (Mentor mode, Set the clock, etc.).	<input type="checkbox"/>
Understands how to change parameter settings (NIBP, EtCO2, SpO2).	<input type="checkbox"/>
Understands purpose of Code Marker.	
Can access data from the code (Print Chart, Print Log, or Transfer Data).	<input type="checkbox"/>
User demonstrates sufficient understanding of device to train other users in its use.	

Preceptor Signature: Signature and Printed Name of Preceptor or other verified personnel who have initialed on this form:		
Initial:	Print Name:	Signature:

**PRECEPTEE STATEMENT AND SIGNATURE:**

I have read and understand the appropriate UC Davis Health Policies/Procedures and/or equipment operations manual, I have demonstrated the ability to perform the verified skills as noted, and I have the knowledge of the resources available to answer questions.

<b>Printed Name</b>	<b>Signature</b>	<b>Date</b>
---------------------	------------------	-------------